# FOLLO STABILIS LEG TRAINER

We congratulate you on choosing a Follo Stabilis product. This instruction information makes it possible for you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use.  
Contents: Check list, unpacking/Installation, safety information, how to use, maintenance instructions, technical specifications and spare parts list

ORDER NO.:	
COLOUR:	
SERIAL NO.:	

<b>FUNCTION TEST:</b>	<input type="checkbox"/> Surface: No damages
<input type="checkbox"/> Bottom frame: Wheels, Screws, plastic parts	<input type="checkbox"/> Labels: Serial number
<input type="checkbox"/> Crank: Securing the pin hole	
<input type="checkbox"/> Top frame: Surface, screws, plastic parts	<b>ENCLOSED:</b>
<input type="checkbox"/> Glide tube: Welding, surface, Silicon grease	<input type="checkbox"/> Instructions
<input type="checkbox"/> Adjusting handle	<input type="checkbox"/> Screws

ASSEMBLED (date/sign.):	
-------------------------	--

END CONTROLLED (date/sign.):	
------------------------------	--

PACKING CONTROL: (date/sign.):	
--------------------------------	--

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Thank you for choosing a Follo Stabilis product. We have prepared this instruction manual to enable you to get the best results from the product. Please read these instructions carefully before assembling and using the trainer.

**Contents:**



- Purpose
- Unpacking
- Maintenance
- Extra equipment
- Recycling
- Safety information
- Guidelines for use
- Troubleshooting
- Technical specifications
- Spareparts list

**THE PURPOSE OF THE TRAINING:**

The leg training machine is designed to improve circulation. It can be used standing on the floor (as shown in the illustration on the cover) while the patient sits on a chair/wheelchair, which has been placed on the floor plates (to keep the machine steady and in position during use). Select a resistance that the patient can keep up for at least 15 minutes. Start pedalling using even movements, extending movements as far as possible. Read off time and distance on the counter.

**SAFETY INFORMATION:**

- Make sure that nothing can be pinched or jammed and that the pedals rotate easily and are properly secured.
- Always test the machine's functions yourself and allow the patient to test all functions under supervision before handing out the product for self-training.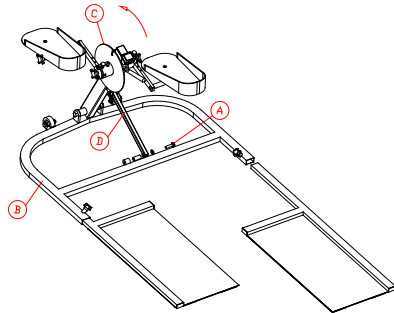
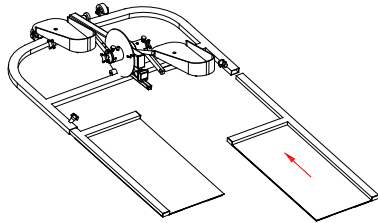


**UNPACKING/ASSEMBLY:**

- Take care when removing the machine from its packaging. Do **NOT** use a knife or other sharp instrument as this may damage the machine. Inspect the machine for any transport damage.
- Place the machine on the floor and mount the floor plates as shown in the left drawing. Remove screw (A). Stand on the frame (B) and lift up the main beam (C) until the angled supporting rod (D) lines up with its attachment point. Replace screw (A). Place two washers between the supporting rod and the attachment point. Do not use a washer on the outside of the supporting rod (see drawing).



**GUIDELINES FOR USE:**

1. Use: → The leg trainer machine is designed for use standing on the floor while the patient sits on a chair/wheelchair, which has been placed on the floor plates. Make sure the machine is properly attached to the floor plates and that the chair/wheelchair is steady.
2. Adjusting resistance: → Adjust resistance using the levers (1). Rotating the levers in a clockwise direction increases resistance, and loosening anti-clockwise reduces resistance.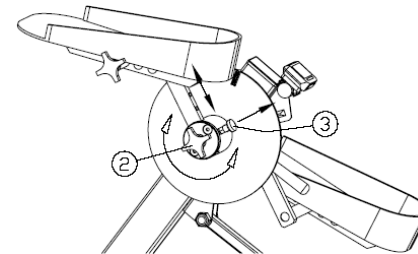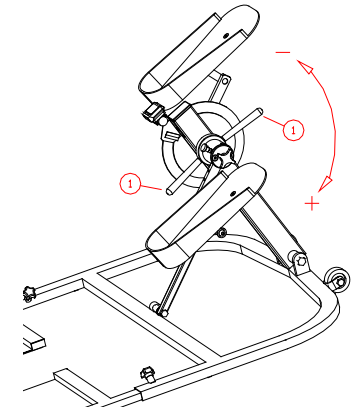

3. Adjusting the pedals: → The crank throw of the pedals may be adjusted by first loosening the knob (2). Pull out the lockpin (3) and push or pull the pedal to the required position. Make sure that the lockpin (3) lines up with a hole and replace the lockpin. Tighten the knob (2).

4. Counter: → The counter (5) is a bicycle computer. It registers when training starts, and is switched on automatically (displaying the last used function). Use the button (6) on the bicycle computer to switch between the different functions: Distance, time, speed, clock and auto (changes regularly between all functions). Holding down button (6) will clear time and distance (not the clock). The computer will switch off automatically when not in use for a period. Read more about the bicycle computer's functions in the separate instructions for use.

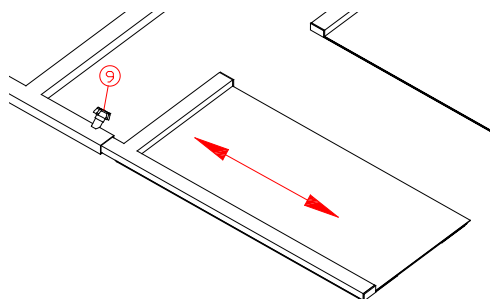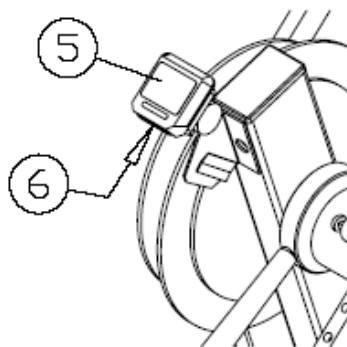5. Floor plates: → Adjustable inwards and outwards by loosening knob (9). Adjust to required position and tighten knob (9).

#### MAINTENANCE:

Inspect the machine regularly for damage, play or missing parts. In the case of damage or other deficiencies, the machine must not be used until it has been repaired.

#### TROUBLESHOOTING:

- Counter / computer:  
If the counter stops working (empty display), check the battery first. If the counter is working, but does not count, check that the black transmitter on the largest nickel plated disc (No. 29 on the parts drawing) is in position. Check also that the opening between the black transmitter and the sensor (immediately below the computer) is correct when they pass each other. The opening must be between 2 and 4 millimetres (maximum 5 millimetres). Adjust the opening by carefully bending the baseplate to which the sensor is attached. See also the folder that accompanied the computer.
- Brake resistance:  
If no brake resistance is felt or the resistance is not functioning properly, the handle may have been adjusted too far in the wrong direction (several turns). Turn the handle clockwise until heavy resistance is felt. Then adjust to the required resistance. If resistance is uneven, inspect the brake lining (No. 30 on the parts drawing). If the brake lining has loosened, it must be replaced.

If these guidelines fail to rectify the situation or if there are other faults or deficiencies, contact your dealer for replacement parts or for service.

**NB! The manufacturer responsibility for CE marking is invalidated if any form of modification is carried out.**

#### EXTRA EQUIPMENT:

- Large pedals
- Handles
- Hooks for wall bars

#### TECHNICAL SPECIFICATIONS:

Height	40 cm
Width:	77 cm
Length (with/without floor plates):	111 / 54 cm
Weight (with/without floor plates):	15,2 / 8,5 kg

#### RECYCLING:

This machine is comprised mainly of steel and plastic. When separating for recycling, the handles and plastic details must be handled as hazardous waste. The remaining parts of the machine can be handled as metal.

The "CE" mark indicates that this product conforms with European standards for health, safety and the environment.

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