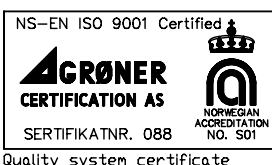
# FOLLO STABILIS TILT

EC DECLARATION OF CONFORMITY FOR  
MACHINERY AND MEDICAL DEVICES

---

Manufacturer: *Follo Industrier AS*

Address: *Brekkeveien 43*  
*1430 Ås*  
*NORWAY*

Herewith declares that: *FOLLO STABILIS TILT (105-8060..)*

- is in conformity with the provisions of the Machinery Directive (Directive 89/392/EEC, 91/368/EEC, 93/44/EEC annex I) and Medical Devices Directive (93/42/EEC annex VII)

Class I.

---

*21.03.02 Ås, Norway*

  
*Ingar Bjerklund*  
*Managing Director*

We congratulate you on choosing a Follo Stabilis product. This instruction information makes it possible for you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use.  
**Contents:** Check list, unpacking/Installation, safety information, how to use, maintenance instructions, technical specifications and spare parts list

ORDER NO.:

COLOUR:

SERIAL NO.:

<b>FUNCTION TEST:</b>	<input type="checkbox"/> Surface: No damages
<input type="checkbox"/> Bottom frame: Surface, screws, plastic parts	<input type="checkbox"/> Labels: Serial number
<input type="checkbox"/> Top frame: Surface, screws, plastic parts	<input type="checkbox"/> Belts
<input type="checkbox"/> Quick release	<input type="checkbox"/> 110 Voltage <input type="checkbox"/> 220 Voltage
<input type="checkbox"/> Tilt function	<input type="checkbox"/> Wheels (directional / total brake function)
<input type="checkbox"/> Raising / lowering	<input type="checkbox"/> Battery back up
<input type="checkbox"/> Workbench	<input type="checkbox"/> Head / foot section angle adjustment
<input type="checkbox"/> Cushions (colour, screws, surface)	<input type="checkbox"/> Labels (lock/release, serial nr, warning)
<input type="checkbox"/> Welding: All connections	<input type="checkbox"/> Enclosed: Instructions

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
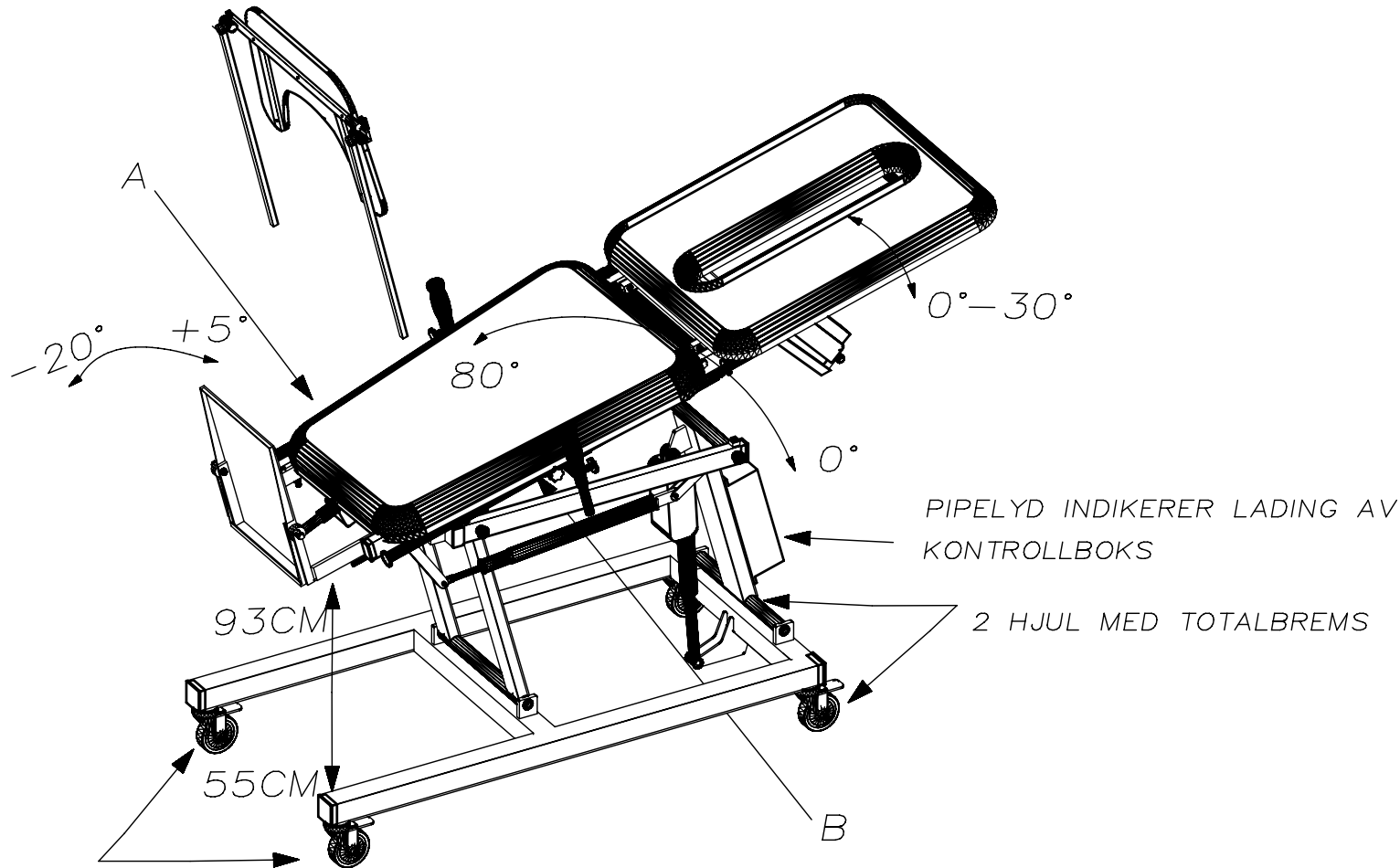
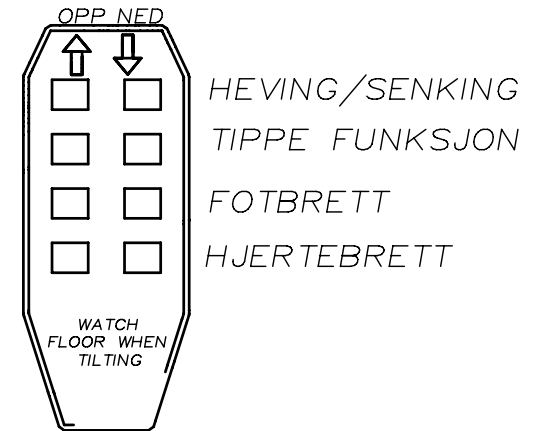
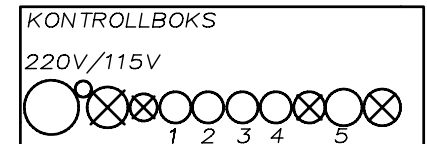
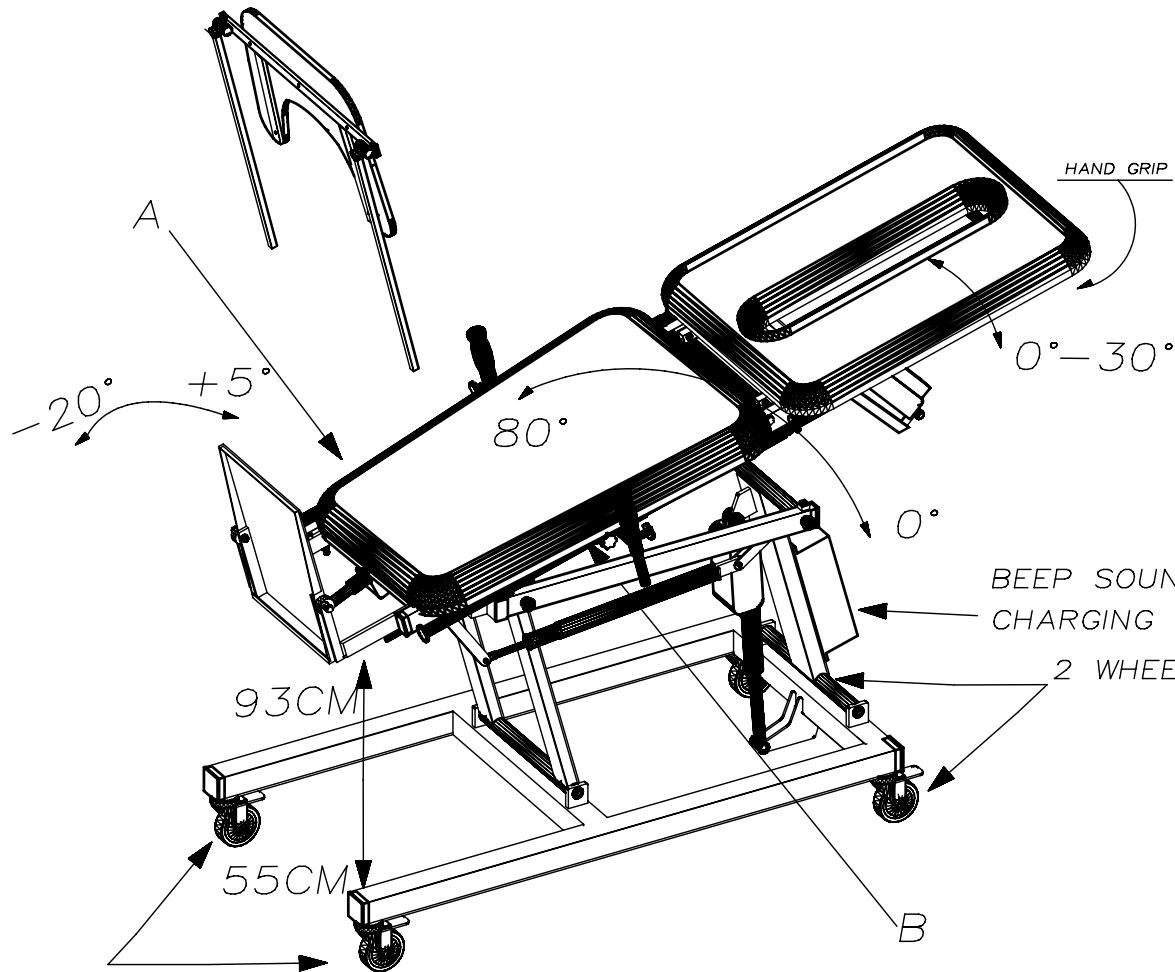
# USE INSTRUCTION

RAISE TABLE TO A MINIMUM OF 10 CM BEFORE FULLY TILTING

## HAND CONTROL

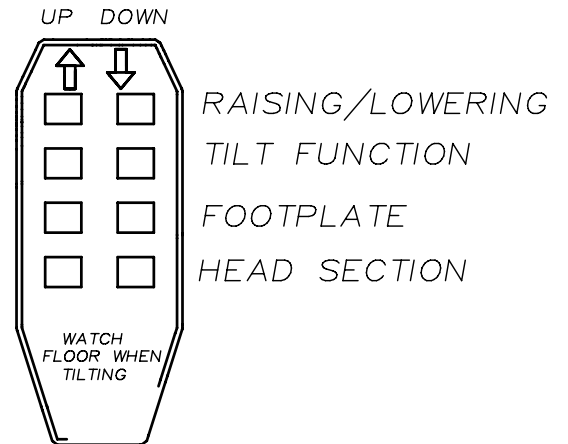
BEEP SOUND INDICATES  
CHARGING OF CONTROL BOX BATTERIES

2 WHEELS WITH ALLROUND BRAKES

CABLE CONNECTING  
CONTROL BOX

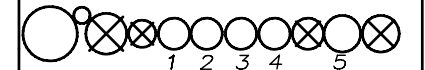220V/115V

1. TILT FUNCTION
2. RAISING/LOWERING
3. FOOTPLATE
4. HEAD SECTION
5. HAND CONTROL

2 WHEELS WITH DIRECTIONAL BRAKES

A - RELEASE HANDLE FOR FOOTPLATE

B - RELEASE FOR QUICK LOWERING

# FOLLO STABILIS EL.TILT

Congratulations on choosing a Follo Stabilis product. This manual enables you to take full advantage of the product from the first day in use. Please read these instructions carefully before faking the product into use. Contents:

- How to use
- Maintenance instructions
- Technical specifications
- Spare parts list
- Safety information

## HOW TO TRAIN:

The EI-tilt is used for training to an upright position. Choose an angle so that the patient can exercise comfortably. Note the angle each session. Try all the functions on a colleague before the EI-tilt is used on a patient.

## SAFETY INSTRUCTIONS:

1. Ensure that nothing gets entangled in the moving parts while the product is used. Be sure that the Velcro is in proper condition and that the belt can be properly fastened.
2. Describe to the patient what the exercise comprises before the treatment.
3. Make slow process with small angular deflections. Keep eye contact with the patient at all times.

## HOW TO USE:

1. The patient may lay down on the bench when it is horizontal. Make sure that the sole of the foot is in contact with the foot plate. Strap on the belts. There are three belts: Two small for the torso and hip, and one large to knees.
2. Adjust for the back rest to a comfortable position. Tilt the bench slowly upwards and adjust the belt if necessary. Check that the feet are resting on the footplate. Tighten or loos the Velcro as necessary.
3. The table can be assembled now if wanted. The table can not be used instead of the upper fastening belt.
4. The angle of the footplate can be adjusted while the bench in a upright position. You can raise and lower the bench also in the upright position. The battery back up enables you to operate the tilt table not being connected to a main power

## MAINTENANCE INSTRUCTIONS:

Check the following at regular intervals (every third month):

1. First clean then lubricate moving parts often by using a little Vaseline oil.
2. Freeplay in such things as straps and handles should be adjusted immediately to avoid further wear and tear.
3. Change worn fixing screws etc. to ensure the equipment functions satisfactorily.

## EXTRA EQUIPMENT:

- Large foot plate

## TECHNICAL SPECIFICATIONS:

Min/max height	55-93cm
Tilt angle	0°-80°
Foot plate	-20°-+5°
Back rest	0°-30°
Four 24V Linak motors	Maximum load 170kg
Weight:	112 kg
Length:	197cm
Width:	70cm

## IMPORTANT:

**AFTER UNPACKING, THE BATTERIES SHOULD BE CHARGED FOR AT LEAST 12 HOURS USING A GROUNDED, 230V OR 11 OV SOCKET. IT IS RECOMMENDED THAT THE BATTERIES BE RECHARGED OVERNIGHT IF A MAINS POWER SUPPLY IS NOT USED. THE ELECTRICAL PLUG MAY REMAIN IN THE SOCKET WITHOUT HARMING THE BATTERIES.**

## QUICK RELEASE TILT FUNCTION:

**USE THE HANDGRIP AT THE REAR OF THE BENCH, AND TURN THE HANDLE WHICH IS ATTACHED TO THE TILT MOTOR. THE TILT FUNCTION CAN BE MOVED BY HAND. RELEASE THE HANDLE AND THE TILT FUNCTION AUTOMATICALLY LOCKS.**

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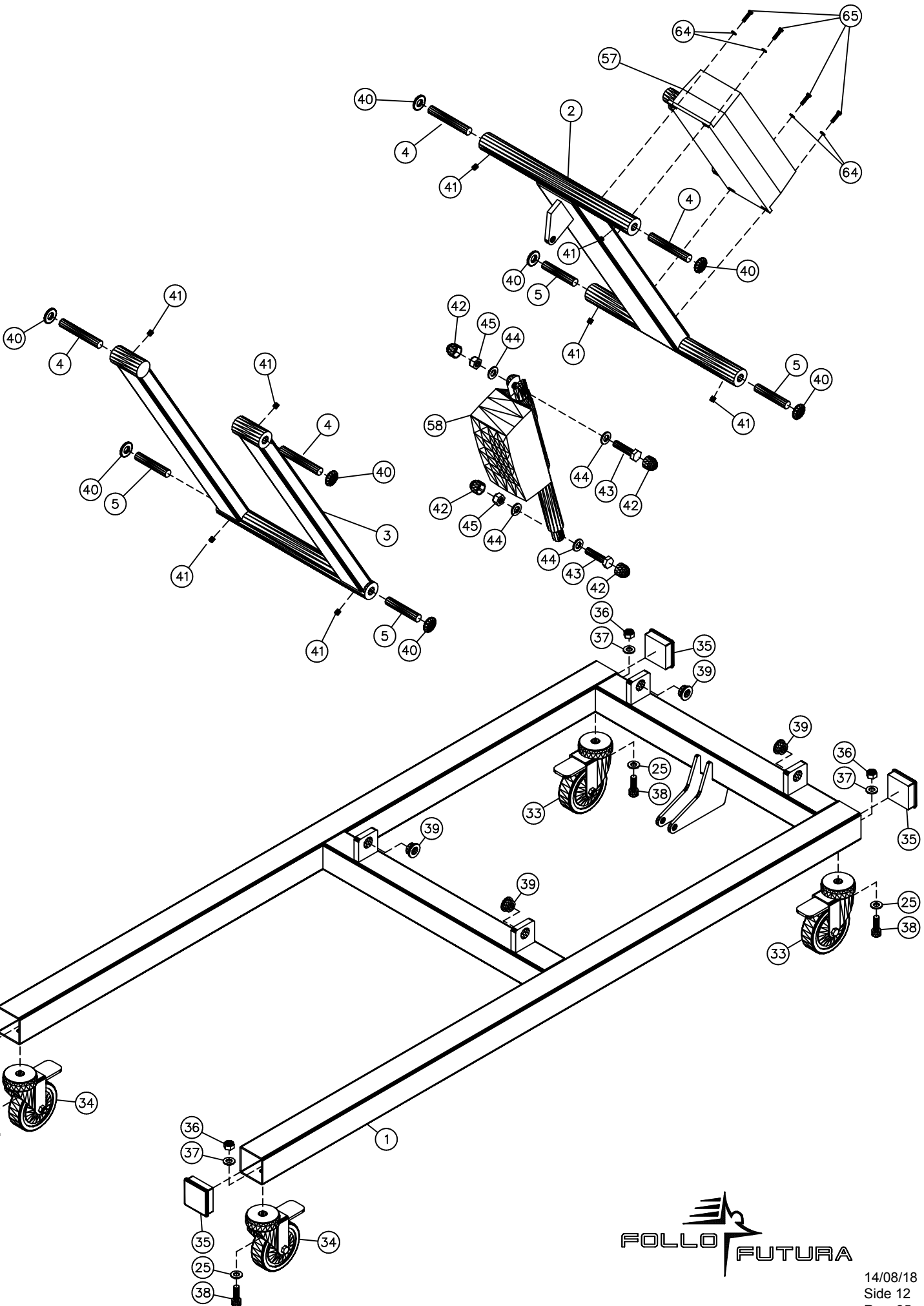
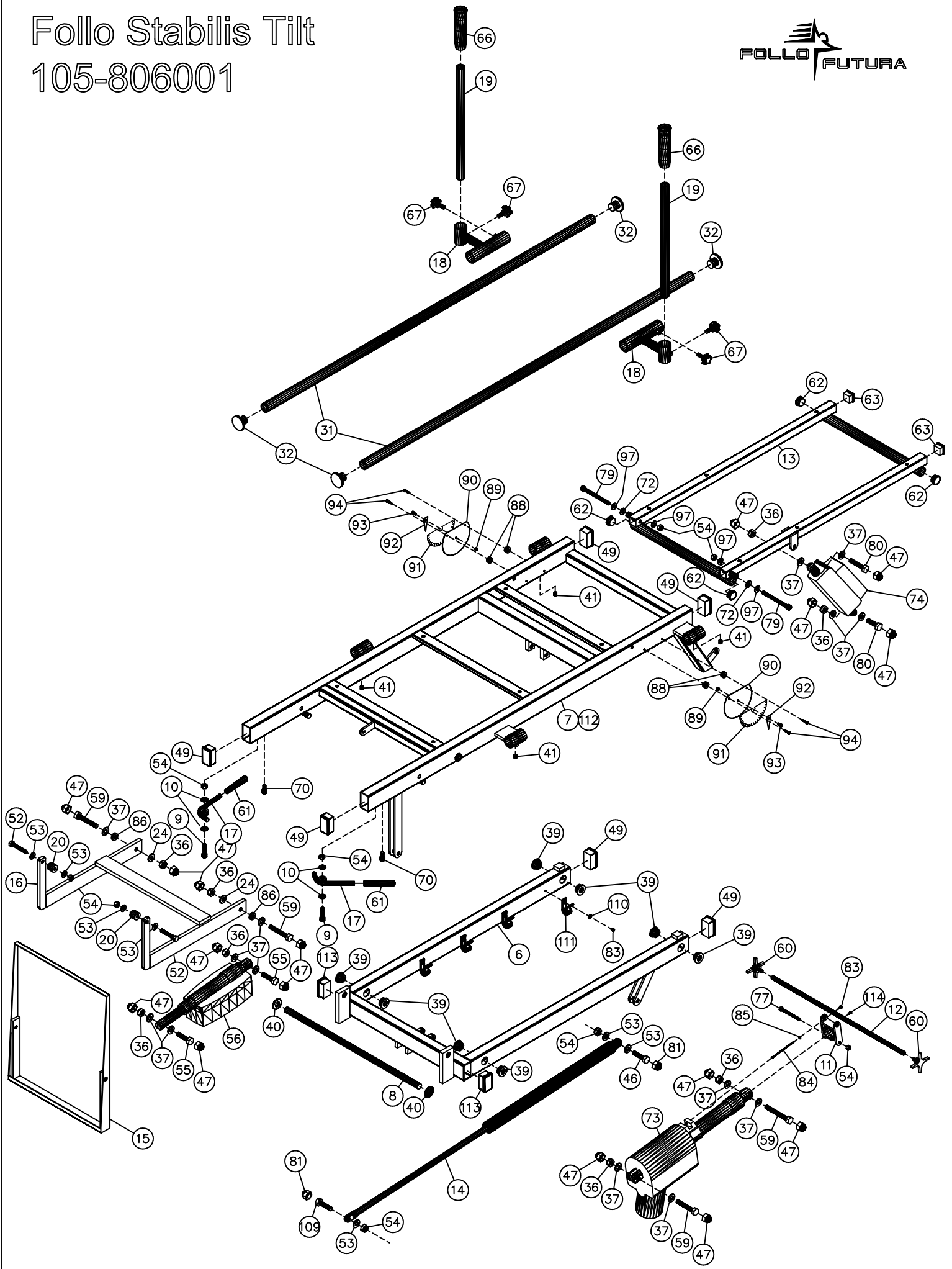
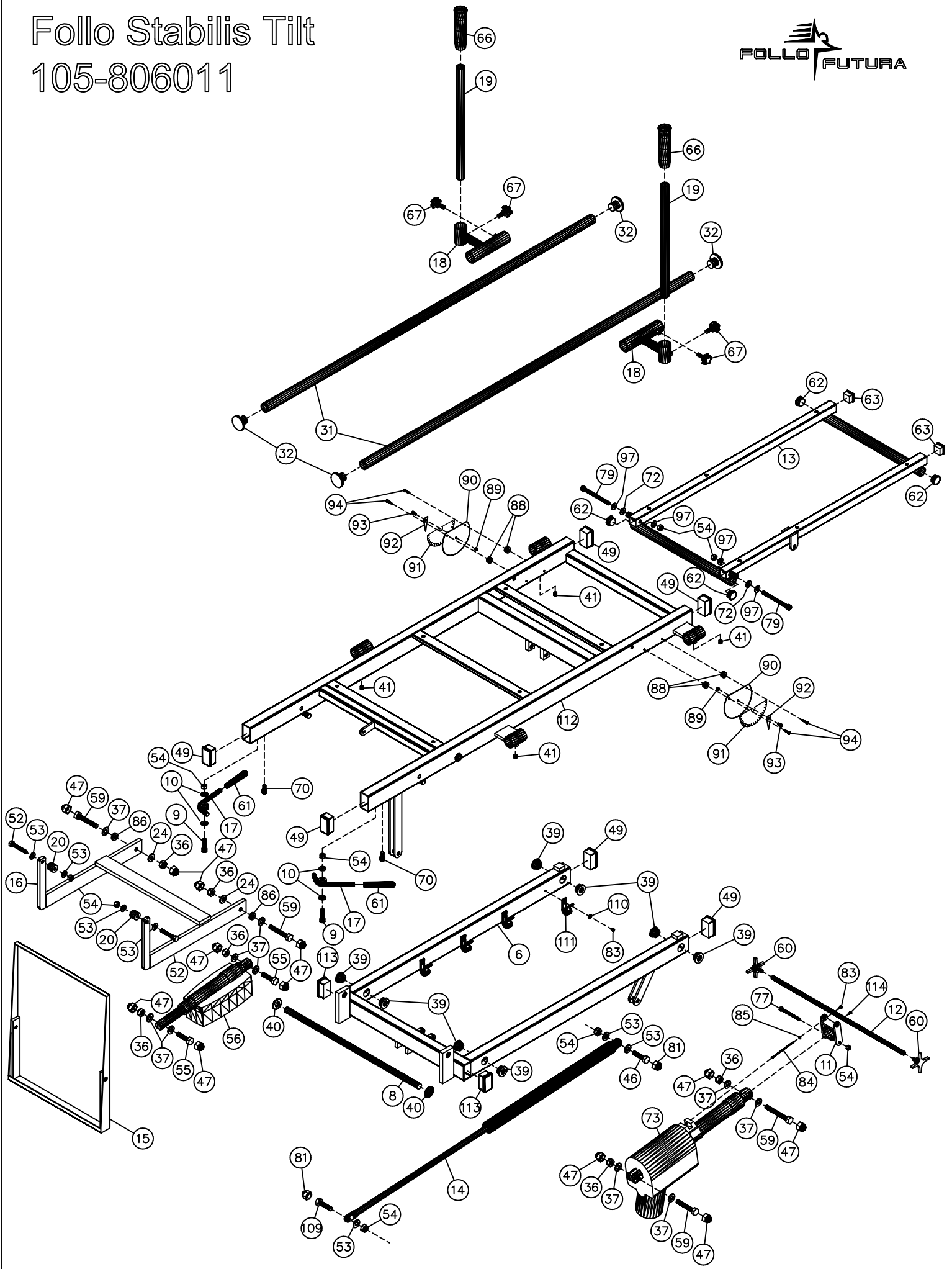
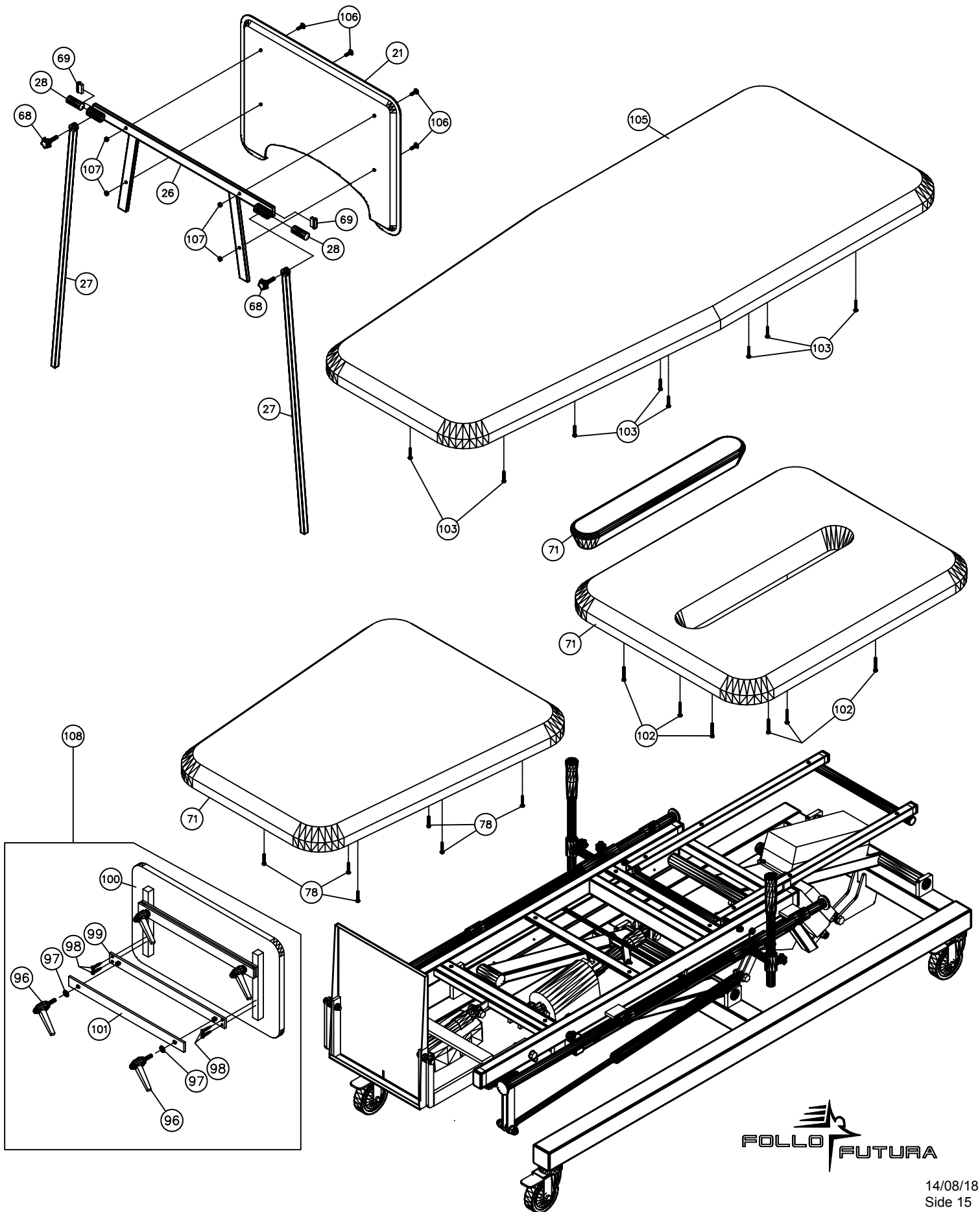
\*) Four different colours.

# Service points

1. Check all screws and nuts on the bench.
2. Check all bolts on the bottom frame, lifting cradles and topframe.
3. Check all star locks.
4. Check the foot plate for cracks.
5. Check the handles on the cushion frame.
6. Make sure the emergency release are working.
7. Check all set screws.
8. Check all motor fittings for cracks.
9. Lubricate all moving points.
10. Need the paint to be improved, use color RAL 9006.
11. Check the belts (two small and one large) for any damage.
12. Moisten a cloth (on the marked commonly used cleaning / disinfectant) and wipe off. NOTE: Alcohol based agent should not be used on plastic covers or cushions.
13. Check the cushions for any damage or cracks.